


## Oral Rinse Collection Instructions For OraRisk® COVID-19 RT-PCR Testing

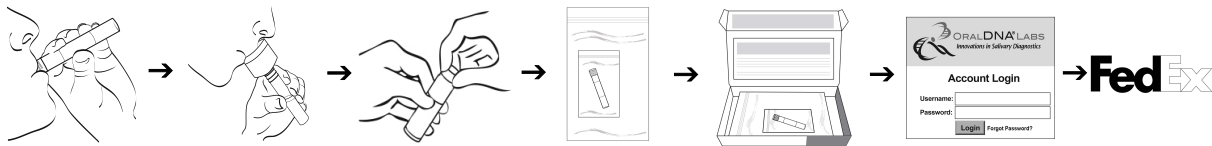

1. **VIGOROUSLY SWISH & GARGLE ALL 5 mL** of saline solution for **AT LEAST 30 SECONDS**
2. **SPIT** saline into funneled collection tube; **DISCARD** funnel
3. **SEAL** collection tube with red cap
4. **PRINT** “Patient Name” & “DOB” on barcode label; **AFFIX** lengthwise onto collection tube
5. **DOUBLE BAG** collection tube using bag with absorbent pad first
6. **PACK** into box; **CLOSE & SEAL** box
7. **LOGIN** to oraldna.com; **CREATE** test order; **PRINT** return shipping label
8. **AFFIX** shipping label to included FedEx Clinical Pak; **CLOSE & SEAL** Pak;  
**SHIP** to OralDNA® Labs

**OralDNA® Labs must receive OraRisk® COVID-19 RT-PCR samples within 72 hours from collection**