


OraRisk® – COVID-19 RT-PCR Oral Rinse + Nasal Swab Collection Instructions - In Office

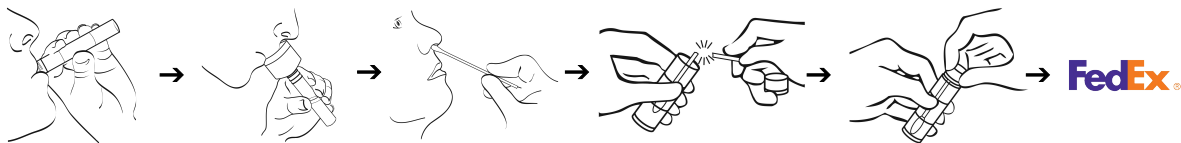

REFRIGERATE GEL PACK FOR 24 HOURS PRIOR TO COLLECTION - DO NOT FREEZE

- 1. VIGOROUSLY SWISH & GARGLE ALL 5 mL** of saline solution for **AT LEAST 30 SECONDS**
- 2. SPIT** saline into funneled collection tube; **DISCARD** funnel
- 3. INSERT** swab tip into a single nostril 1 inch beyond opening and swab the inner surface for **AT LEAST 30 SECONDS**
- 4. INSERT** swab into collection tube; **BEND** swab handle until it breaks at score mark; **SEAL** collection tube with red cap
- 5. PRINT** "Patient Name" & "DOB" on barcode label; **AFFIX** lengthwise onto collection tube; **DOUBLE BAG** tube using bag with absorbent pad first; **PACK** into foil bag along with **REFRIGERATED** gel pack; **SEAL** foil bag; **PACK** into box; **CLOSE & SEAL** box
- 6. LOGIN** to oraldna.com to **CREATE** test order and return shipping label; **PRINT** label
- 7. AFFIX** shipping label to included FedEx Clinical Pak; **SHIP** to OralDNA® Labs